# Pluggierz Uni-Fit

## Manual Handleiding Handbuch Manuale

-  
-  

## pluggierz earplugs

Your ears love us.



-  
-  

- Do not wear Pluggierz Pro if there is a risk of the connecting cord getting stuck behind moving (machine) components.
- Pluggierz Road; some Countries or states may prohibit using ear plugs while driving – check your local and state’s driving and traffic laws.
- This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.

If Pluggierz are inserted incorrectly or if the warnings in the operating instructions are not followed they may not offer full protection Comfoor B.V. Pluggierz can never be held responsible for improper or incorrect use or any failure to comply with the warnings stated above. For more information, please go to **www.pluggierz.com**.

- Check to see if the Pluggierz are clean and undamaged.
- Hold the top of your auricle (ear-shell) and gently pull it upwards and backwards.
- Insert the hearing protector into the ear canal deep enough as comfortably possible, to seal the ear canal.
- Let go of the auricle and gently and confidently push the hearing protector once again.

Pluggierz Uni-Fit Music Premium comes with one set of high-quality filters. These are fitted as standard in the large earplugs. If

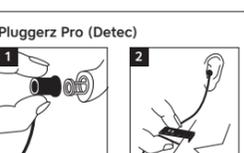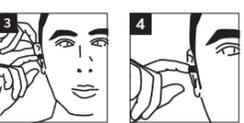### ENGLISH

Pluggierz protect against the risk the risk of hearing damage as a result of noise. Pluggierz are reusable hearing protectors made from a strong, skin-friendly silicon material.

**Maintenance**
Pluggierz are easy to clean with water and soap, which ensures optimal hygiene. For cleaning of the Music Premium remove filter before cleaning the earplug. Dry them well before inserting them into your ears or placing them back in the pouch.

- Warning**
  - Pluggierz only provide optimal protection when worn continuously in a noisy environment.
  - Do not swallow.
  - Keep out of reach of children till 3 years old.
  - In case of any complaints, please contact us or consult a specialist or physician.
  - Pluggierz must be cleaned in accordance with the instructions of the manufacturer, such as tearing, hardening or adhesion of the material before use. Also make sure that the filter is in place.
  - Never use Pluggierz when damaged, if the material shows signs of decay or if the filter is not in place.
  - Pluggierz must be cleaned in accordance with the instructions of the manufacturer.
  - The sudden, quick removal of the earplug can damage the eardrum.
  - Do not stick sharp objects into the channel or filter of the earplugs.
  - Music premium filters should be kept dry at all times.
  - Do not use the Pluggierz Swim at a depth of more than 3 meters under water. It is strongly advised not to use them while jumping or diving.

- Do not wear Pluggierz Pro if there is a risk of the connecting cord getting stuck behind moving (machine) components.
- Pluggierz Road; some Countries or states may prohibit using ear plugs while driving – check your local and state’s driving and traffic laws.
- This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.

If Pluggierz are inserted incorrectly or if the warnings in the operating instructions are not followed they may not offer full protection Comfoor B.V. Pluggierz can never be held responsible for improper or incorrect use or any failure to comply with the warnings stated above. For more information, please go to **www.pluggierz.com**.

**How to insert Pluggierz Uni-Fit**
In this package, you will find 2 sizes of Pluggierz earplugs. First try the smaller size earplug following the instructions below. If the larger earplug is hard to insert or if it feels uncomfortable, you can try the smaller size.

- Check to see if the Pluggierz are clean and undamaged.
- Hold the top of your auricle (ear-shell) and gently pull it upwards and backwards.
- Insert the hearing protector into the ear canal deep enough as comfortably possible, to seal the ear canal.
- Let go of the auricle and gently and confidently push the hearing protector once again.

Pluggierz Uni-Fit Music Premium comes with one set of high-quality filters. These are fitted as standard in the large earplugs. If

the large earplugs do not fit, you can easily change the filters to the small earplugs. **Warning:** without filters the earplugs do not protect. Never use the earplugs without filters.

Ensure that earplugs are comfortably inserted with a proper seal. If you think that noise is too loud for comfort, the product might not be sealed properly. If you cannot find a proper seal, you may need to consult a physician.

Protection is only guaranteed if the ear protector is inserted correctly.

After use, always store the Pluggierz in the supplied storage case or pouch.

*The European type-examination has been carried out by PZT 1974, An der Junkerei 48F, 26389, Wilhelmshaven, Germany and IFA 0121, Alte Herrstrasse 111, 53757 Sankt Augustin, Germany.*

*U.S. EPA Regulation 40 CFR Part 211 Declaration of Conformity: Comfoor B.V. hereby declares that Pluggierz comply with CE Regulation (EC) 2016/425. The complete text of the EC Declaration of Conformity is available on our website **www.pluggierz.com**.*

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For U.S. E.P.A. Regulation statistics see **www.pluggierz.com** Made in Holland

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